## 6634 NOMBIX

### Mascarilla Higiénica

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## 6634 NOMBIX

### Hygienic Mask

**Hygienic triple-layer mask** with heat-sealed finish, elastic fastening and adaptable nose clip for a tight fit.

#### Composition:

3 Layers:

- Outside 1: Non-Woven (Polypropylene)
- Middle layer: Meltblown
- Outside 2: Non-Woven (Polypropylene)

#### Specifications:

- In compliance with **UNE 0064-1:2020**
- Bacterial filtration efficiency (BFE): **≥ 95%**
- Breathability (differential pressure): **< 60 Pa/cm<sup>2</sup>**
- Non-reusable

#### Recommended for:

- Keep the air free of pathogens generated by the mask wearer
- Personal use
- Adult users

#### Available documents:

- Declaration of Conformity
- Test Report of Bacterial Filtration Efficiency and Breathability according to EN 14683:2019
- REACH certification

#### Logistics information:

- Presented in master cartons of 3.000 pcs
- Inner boxes of 50 pcs
- Bags of 50 pcs
- Master carton size: 41 cm height / 50 cm width / 52 cm length
- Weight: 12.8 kg/carton
- Quantity per pallet: 48.000 pcs; 16 cartons

## RECOMMENDATIONS FOR USING AND FITTING THE MASK

The use of hygienic masks does not exempt the user from the application of barrier actions, complemented by the essential, social distancing measures, in accordance with the recommendations of the health authority.

### MASK WEARING

- 1 Wash hands with soap and water or rub them with a hydroalcoholic solution before handling the mask.
- 2 Identify the top of the mask and adjust the fastening straps behind the ears.
- 3 Adjust the nose clip until a firm, yet comfortable fit, is achieved.
- 4 Place the lower part of the mask under the chin and check that it is properly adjusted. To accomplish this, it is necessary to check the good seal as well as not feeling respiratory discomfort.
- 5 Once adjusted, do not touch the mask with your hands. If you need to touch the mask, it is advisable to wash your hands first with soap and water or rub them with a hydroalcoholic solution.

### MASK REMOVAL

- 1 Wash hands with soap and water or rub them with a hydroalcoholic solution before handling the mask.
- 2 Do not touch the front of the mask.
- 3 Remove the mask by pulling the elastic bands around the ears.
- 4 Place the mask to be disposed of inside a dedicated container, preferably with a lid and a non-manual control. It is recommended to use a double bag as a safety measure in case the outer bag breaks. It can also be disposed of in biological waste containers.
- 5 Wash hands with soap and water or rub them with a hydroalcoholic solution after handling the mask.

For reasons of comfort and hygiene, it is advisable not to use the mask for more than 4 hours. In case it gets wet or deteriorates from use, it is recommended to replace it with a new one.

During this time, it can only be used several times if it is removed according to the instructions described above, stored temporarily or hung up to provide as little contact as possible, and then replaced according to the instructions. Nor should it be placed in a waiting position on the forehead or under the chin during and after use.